

NEW EQUIPMENT

COCOON CONVECTIVE PATIENT WARMING MACHINE

Exposure

This device replaces the old Bear Hugger that has moved on from this world. It provides a continuous method of warming patients.


The unit has no Battery backup so must be plugged in to power to be used.

Adult and Paediatric warming blankets should be stored in the basket on the stand with the warmer

1. Unfold blanket and place over the patient
2. Connect delivery hose by pushing plastic fitting into the air inlet port
3. Secure with ties by wrapping first underneath the hose and then tie at the top
4. If you want to warm patient quickly press the 46°C button. Alternatively choose your output temp.
5. Wipe over with cleaning wipes when finished

Fault Alarm—Notify Biomed

Filter Status Light—Notify Biomed

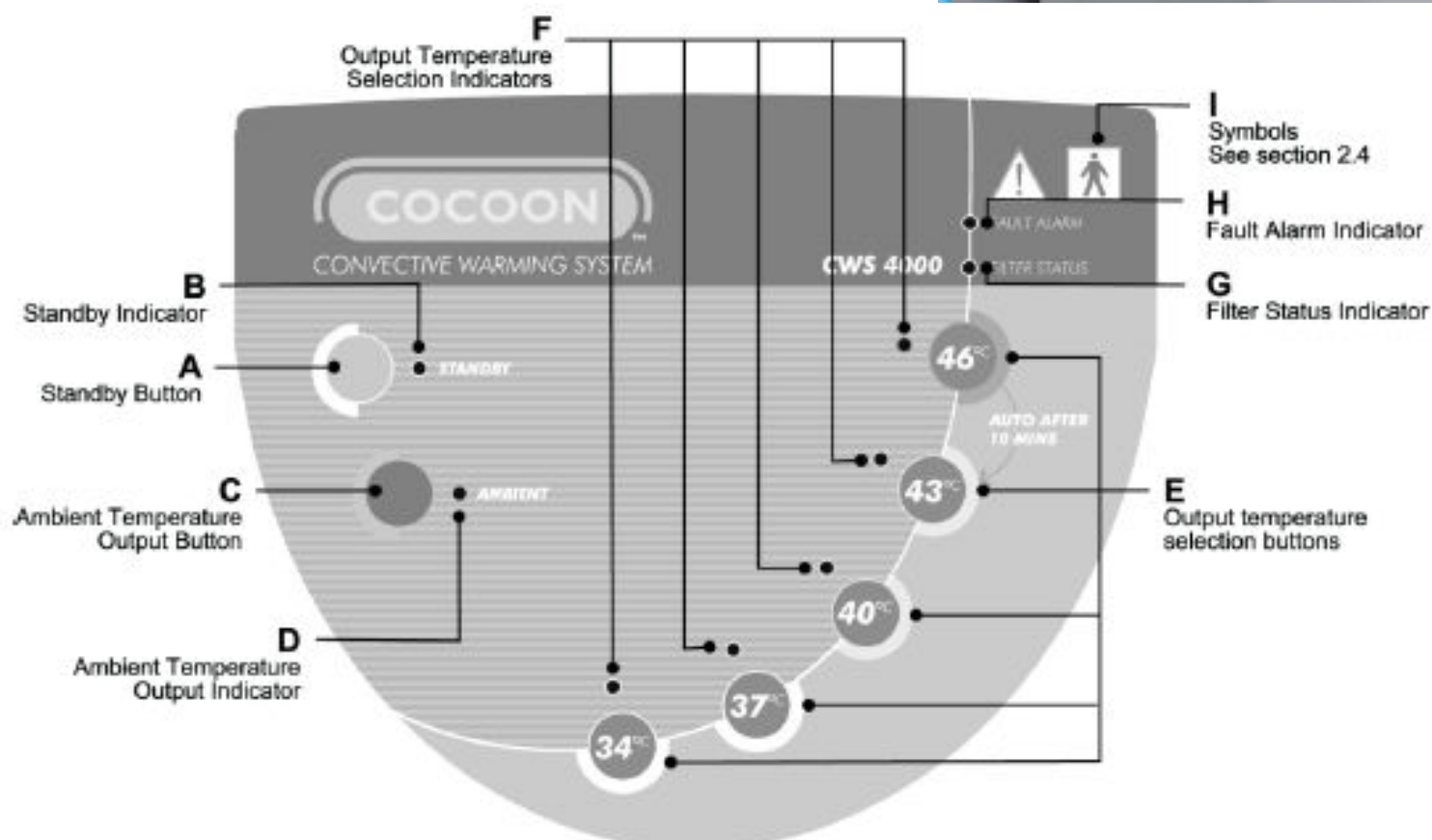

If you have any questions please contact ED CNE—Noelene Williams